C | S

47100  
MITELLA  
MESH

## 47100 ARMSLINGA NÄT

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## 47100 MITELLA NET

Carefully read this instruction sheet before use!

# ENGLISH

**Intended use/Indications:** Intended for temporary off-loading for shoulder and arm. Indications: Post-trauma and post-surgery conditions.

**Contra indications:** No known.

**Features:**

The shoulder strap can be adjusted stepless with the Velcro strap. The double D-rings allows the sling to be used with or without cast. The sling also have a thumbstrap.

**Adjustments:**

This product should only be altered by a medical professional person.

**Washing Instruction:**

Close all Velcro por to laundering. Machine wash in 60°C (140°F). Dry flat. Do not tumble dry or dry clean.

**Material Specifications:**

Main material: 100% Polyester

Velcro: 100% Nylon

Thumb strap, shoulder- and edgingtape:

100% Polyester.

D-ring: 00% POM


This product is not flammé retardant.

**Who to Contact:**



If any problems occur, remove the support and contact your fitter or medical practitioner.

**Time of use:**

Follow the medical practitioners guidelines for use and limitations. This product is recommended for one person use only.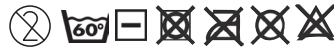

## Instructions for use

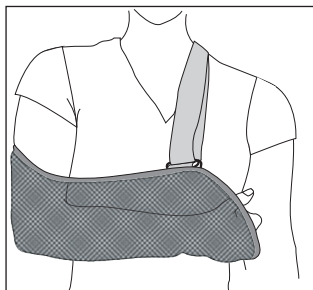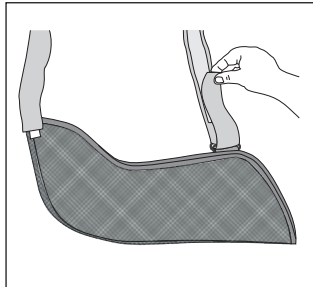1. Check that the product is complete and without damage.
2. Position the arm into the sling and place the strap over head so the strap comfortably rests on the opposite shoulder.
3. Adjust the strap length so the elbow angle is approx. 90°.
4. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38