

# Elements Body

Dynamic compression for optimal functionality



# Sensory and proprioceptive feedback for the whole body!

**Elements Body** is a dynamic textile orthosis made of lycra fabric, composed of various panels. It is designed to give constant proprioceptive feedback through pressure and resistance.

The function of the suit is to improve posture and stability, this is achieved by providing pressure to the body through a snug Made To Measure (MTM) fit. By increasing the deep sensibility, proprioceptive feedback is increased.

The proprioceptive feedback may also improve the patient's fine motor skills and functional performance.

Various reinforcements can be selected to optimise flexion, extension, pronation and supination positions.



---

## How should Elements Body be used?

Once the garment has been fitted, it is advised that tolerance is increased gradually.

When the recommended minimum wearing regime of 6 hours is achieved the maximum time should not exceed 8 hours a day.

Elements Body should not be worn through the night when the patient is sleeping.

## Benefits with Elements Body

- Increased proprioceptive feedback
- Improved body awareness
- Increase of fine motor skills
- Improve posture
- Improved movement patterns

## Indications

Elements Body is suitable for the following neurological diagnoses presenting with specific symptoms.

- Cerebral Palsy: Hypotonia, Spasticity, Ataxia
- Multiple Sclerosis, Stroke & Acquired Brain Injury: Spasticity, Ataxia
- Muscular Dystrophy, Acquired Spinal Cord Injury and Down's Syndrome: Hypotonia
- Athetosis: Spastic Athetosis

# Trial garments for informed assessments

Our comprehensive order forms, guidelines, and measurement forms are conveniently available for download directly from our website. Streamlining the process, this online resource ensures ease of access to essential documentation, facilitating a seamless experience for practitioners and healthcare professionals.



*Scan or click for more product information, sizes and measurements!*



## Elements Body Trial Sleeve

The Trial sleeve has been designed to be used in the assessment process for Elements Body MTM upper limb garments. It enables the clinician to evaluate the user's response to the application of a dynamic textile orthosis in preparation for a made-to-measure upper limb orthosis. The Trial sleeve is only to be used for an assessment period of 2-4 weeks to assist the clinician in identifying the clinical effectiveness and tolerance prior to the provision of an Elements Body MTM Garment.



## Elements Body Trial Lumbar Wrap

The Trial Lumbar Wrap is an evaluation tool to be used in the assessment process for Elements Body. It enables the clinician to evaluate the patient's response to the application of a dynamic textile orthosis in preparation for a made-to-measure garment. The Trial Lumbar Wrap is intended for short-term use for a maximum of two weeks.



## Elements Body Easy On

Easy on is intended to be used as an aid when donning an Elements Body or HyMo orthosis. Wearing the product when donning will reduce the friction and make it easier to put on the orthosis. When the Elements Body/HyMo orthosis is on, Easy on is removed.



## Support for Better Life

Everyone should be able to live their life to the fullest, regardless of their mobility challenges. With innovative solutions developed in close collaboration with healthcare professionals and patients, we strive to provide Support for Better Life.

# allard

Allard International

Karbingatan 38, 254 67 Helsingborg, Sweden | +46 42 25 27 00 | [info@allardint.com](mailto:info@allardint.com) | [allardint.com](http://allardint.com)