**BROMSA
28240**

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6. Etterjuster slik at ortosen sitter behagelig.

Indications / Intended Use

This product is intended to be used as prevention or treatment for lateral epicondylitis, overmobility, and unspecific pain in the wrist.

Contraindications

No known.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check you skin on a daily basis.

Material Specifications

This orthosis is made from polyester, cotton, elasthane and with a filling of 100% polyurethane. This product is not flame retardant. Latex free.


Washing Instruction

Close all Velcro prior to laundering. Remove the plastic stays prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.


Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instruction for use

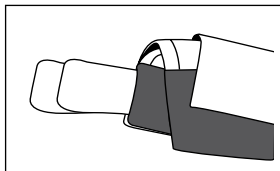
1. Check that the support is complete and without damage.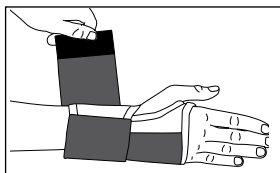

2. This support has two plastic stays in the pocket. Use one stay for mild resistance and two for increased resistance. Contact the prescriber or fitter when in doubt.

3. Pull the support on like a glove. The stay shall be placed on the back of the hand.

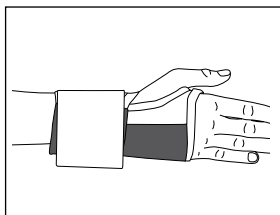4. Attach the inside Velcro.

5. Pull the strap around the wrist and attach.

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