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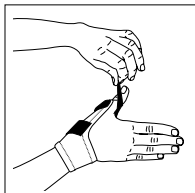
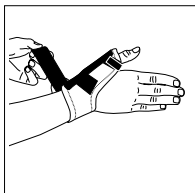
# ENGLISH

## 46028 VISSION THUMB ORTHOSIS

Carefully read this instruction sheet before use!

### Indications / Intended Use

This product is intended to be used for instability in the thumb joints (CMC- or MCP joints), inflammation, joint stress and for rheumatic degenerations.

### Contraindications

No known

### Adjustments

This product should only be altered by a medical professional person.

### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

### Material Specifications

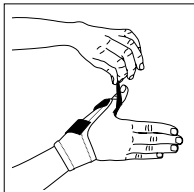
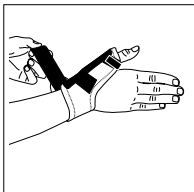
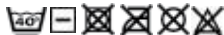
This orthosis is made from cotton, nylon, polyester, lycra and spandex. Spoon made in POM. This product is not flame retardant. Latex free.

### Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

## Instructions for use

1. Check that the support is complete and without damage.
2. Open the Velcro for the wrist.
3. Place the orthosis over the thumb.
4. Pull around the wrist and secure the Velcro.
5. Adjust for comfort.

If needed, shape the stay on the thumb side.

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