**35300
SELECTION
CHILDRENTHUMB**

35300 SELECTION TUMORTOS BARN

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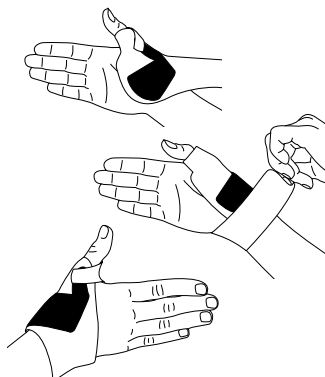
Carefully read this instruction sheet before use!

Intended Use

To guide the thumb and hand into a more functional position to facilitate the gripping ability.

Indications

Brachial plexus birth palsy (BPBP), Spastic hand in children with Cerebral Pares. Arthrogyposis multiplex congenital children, Rheumatic problems.

Contraindications

No known

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

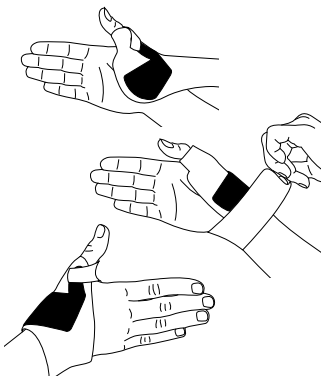
This orthosis is made from polyester, nylon and elastan. This product is not flame retardant

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the support is complete and without damage.
2. Attach the thumb strap.
3. Place the opening over the thumb with the orthosis towards the back of the hand.
4. Pull the strap around the wrist and secure.
5. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38