47014
VISSION
NECK COLLAR

SVENSKA

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ENGLISH

47014 VISSION NECK COLLAR

Carefully read these instructions before use!

Indications / Intended Use

This soft neck collar is intended to provide relief to the neck for those with muscle pain or other pain relating to rheumatism. This product is recommended for single person use only.

Contraindications

None known.

Limitations

This product should only be altered by a medical professional person. The prescriber will determine the duration and conditions for use.

Warning

Do not pull too tight, as this can cause poor circulation and/or skin irritation. If you feel any irritations or other problems, take it off immediately. Check skin on a daily basis.

Material specifications




This product is made from Polyurethane, Polyamid, Spandex and Nylon. Does not contain Latex. This product is not flame retardant.

Washing instructions

Close all Velcro prior to laundering. Hand wash with mild detergent. Dry flat. Press the water out from the foam. Do not twist or squeeze to remove excess water. Do not tumble dry or dry clean. This product is not flame retardant.


Who to contact

Contact your fitter or store where purchased if you have questions regarding the use of the product.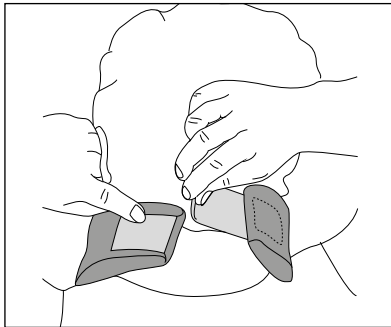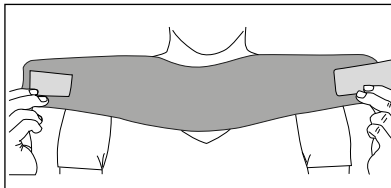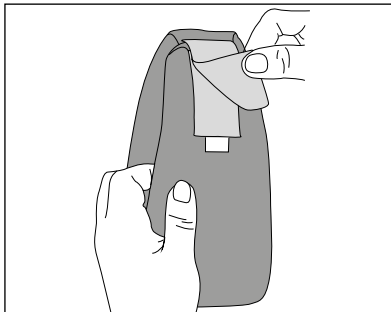


Instructions for use

1. Check that the product is complete and without damage.
2. Open the Velcro.
3. Position the collar so the closure is at the back of the neck. In some cases your medical professional may require the closure to be in the front. Please check with prescriber if uncertain.
4. Close the collar at the back of the neck.
5. Collar should fit snugly but not restrict circulation.
6. Adjust for comfort.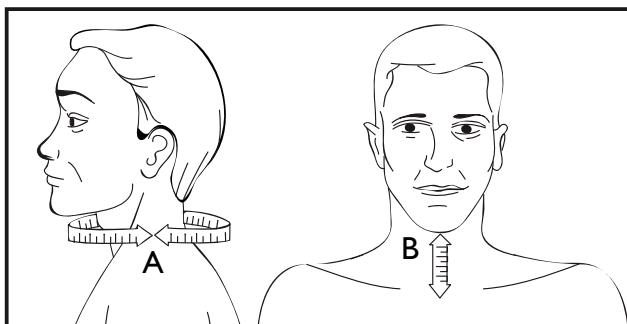

Item No.	Description	A Circumference	B Height
47014 0010	VISSION Neck Collar Shaped, black	32-48 cm	6 cm
47014 0011	VISSION Neck Collar Shaped, black	32-48 cm	8 cm
47014 0012	VISSION Neck Collar Shaped, black	32-48 cm	10 cm
47014 0013	VISSION Neck Collar Shaped, black	32-48 cm	12 cm

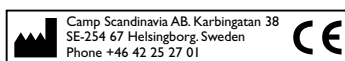SVERIGE	DENMARK	SUOMI	NORGE	UK/IRELAND	USA	INTERNATIONAL
Tel 042-25 27 01	43 96 66 99	09 350 76 30	23 23 31 20	+353 42 932 8177	888-678 6548	+46 42 25 27 01
Fax 042-25 27 25	43 43 22 66	09 350 76 388	23 23 31 21	+353 42 932 8182	800-289 0809	+46 42 25 27 25
info@camp.se	info@camp.dk	info@camp.fi	info@camp.no	info@allarduk.co.uk	info@allardusa.com	info@allardint.com

Support For Better Life • www.allardsupport.com